HIPPEX PLUS

EQUINE SEMEN EXTENDER

HIPPEX PLUS, an equine semen extender for fresh use and cooled transport.

Instructions for use.

Have beforehand enough water (distilled or demineralized water) stored in an incubator that is set at 30°C. For best results, prepare the HIPPEX PLUS extender just before collecting the semen. Prepare 250 ml water at 30°C in a glass beaker or bottle. Add the HIPPEX PLUS powder to the water. Stir the solution slowly until the powder is completely dissolved. Then put the HIPPEX PLUS extender in the incubator (set at 30°C) until use.

It is recommended to dilute the ejaculate to a sperm cell concentration between 25 and 50 million cells per ml. A concentration far above 50 million per ml will jeopardize the quality of the diluted semen during storage. It is also recommended to dilute the ejaculate by adding at least 2.5 volumes of extender to 1 volume of ejaculate to assure that sperm quality is maintained for at least 48 hours.

Most stallions produce ejaculates with sufficient concentration of sperm cells (more than 100 million cells per ml) to be able to follow both recommendations mentioned. However, when the ejaculate sperm cell concentration is far below 100 million cells per ml, then it is recommended to centrifuge the diluted semen for concentrating the cells.

Immediately after dilution, the tubes with diluted semen may be passively cooled (in the dark) at room temperature or actively cooled in a controlled climate box set at 15-17°C for about an hour. Put the tubes with diluted semen thereafter in a refrigerator, preferably on a rocking table. Unused HIPPEX PLUS medium can be stored in the refrigerator for later use, but it is recommended to use it within 5 days.

Freezing of smaller quantities of HIPPEX PLUS extender.

If you often need small quantities of extender (for example 50 ml), you may freeze portions of the extender in plastic tubes. The tubes can then be stored in the freezer until use. Just before collecting the semen you can thaw the desired quantity of extender.

In that case, dissolving the HIPPEX PLUS powder can then best be done in 250 ml of water with a temperature of 20°C. After dissolving the powder, wait 15 minutes, then stir or shake one last time and finally fill the plastic tubes with the HIPPEX PLUS extender and store in the freezer at – 20°C.

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