## **HIPPEX, an equine semen extender for fresh use and cooled transport.**

### **Instructions for use.**

***For the little jars with 22.5 gr HIPPEX powder:*** Have beforehand enough water (distilled or demineralized water) stored in an incubator that is set at 30°/37°C. For best results, prepare the HIPPEX extender just before collecting the semen. Prepare 250 ml water at 30°/37°C in a glass beaker or bottle. Add the HIPPEX powder to the water. Stir the solution slowly (or use a magnetic stirrer) until the powder is completely dissolved. Then put the HIPPEX extender in the incubator (set at 30°C) for about 30 minutes until use.

***For the brown bottle 300 mL that contains 22.5 gr HIPPEX:*** Add carefully 250 mL distilled or demineralized water at 30°-37°C into the bottle. Close the bottle and shake in a gentle way until all the HIPPEX is completely dissolved. Place the bottle in the incubator at 30°/37°C for about 30 minutes until use. If possible, gentle shake the bottle from time to time or place the bottle on a roller bank.

When the ejaculate sperm cell concentration is far below 100 million cells per ml, then it is recommended to centrifuge the diluted semen for concentrating the cells by adding **HIPPEX CENTRIFUGATION MEDIUM** in a 1:1 dilution.

***Recommendations:*** It is recommended to dilute the ejaculate to a sperm cell concentration between 25 and 50 million cells per ml. A concentration far above 50 million per ml will jeopardize the quality of the diluted semen during storage. It is also recommended to dilute the ejaculate by adding at least 2.5 volumes of extender to 1 volume of ejaculate to assure that sperm quality is maintained for at least 48 hours.

Immediately after dilution, the tubes with diluted semen may be passively cooled (in the dark) at room temperature or actively cooled in a controlled climate box set at 15-17°C for about an hour. Put the tubes with diluted semen thereafter in a refrigerator, preferably on a rocking table.

Unused HIPPEX medium can be stored in the refrigerator for later use, but it is recommended to use it within 5 days.

### **Freezing of smaller quantities of HIPPEX extender.**

If smaller quantities of extender (for example 50 ml) are needed, you may freeze portions of the extender in plastic tubes. The tubes can then be stored in the freezer until use. Just before collecting the semen, you can thaw the desired quantity of extender.

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